

PACKAGE INSERT

HYDRESS™ Hydrogel Dressing

Product Description: HYDRESS™ Hydrogel Dressings are sterile dressings that are 0.055 inches thick and consist of De-ionized water, Polyethylene Oxide, Methylparaben, Propylparaben and Citric Acid. The dressings are available in the following sizes: 2 X 2 inches, 2 X 3 inches, 3 X 3 inches, 4 X 4 inches and 4 X 5 inches.

Indications: HYDRESS™ Hydrogel Dressing is intended for use on skin ulcers, minor wounds, scrapes, minor burns and partial thickness wounds.

Directions for Use: HYDRESS™ Hydrogel Dressings should be placed on the wound and bandaged as necessary. Do not re-sterilize.

Remove dressing from package and remove the blue plastic sheet from dressing. The second plastic sheet (on the outer surface of the dressing) may be left in position to conserve moisture if the wound is dry but if the wound produces more fluid than the dressing can absorb, the second sheet should be removed to allow evaporation through the gel and prevent skin softening due to excess fluid.

- Apply exposed side of dressing to skin.
- Bandage to hold dressing in place.
- Keep wound moist.
- Change dressing daily.
- If dressing should dry on wound, remoisten with sterile water before removing.
- Dressing may easily be cut to size.

Warnings: Do not use if package is damaged. If wound shows signs of infection, wound color or and/or odor change, or does not show any signs of healing, consult your doctor immediately. Keep this product out of the reach of children.

Ingredients: De-ionized Water, Polyethylene Oxide, Methylparaben, Propylparaben and Citric Acid.

Contraindications: Do not use on moderate to heavy draining wounds. Do not use on serious or chronic wounds. Not to be used by persons with a known sensitivity to hydrogel or similar polymer materials.

Interactions: There are no known interactions.

Precautions: HYDRESS™ Hydrogel Dressing is for external use only. Single use only.

Adverse Reactions: There are no known.

Storage: Store HYDRESS™ Hydrogel Dressings in sealed packaging at room temperature. Avoid excessive heat or humidity.

How Supplied: HYDRESS™ Hydrogel Dressings is supplied in the following sizes:

Product No.	Size	How Supplied
AB-200200-002	2" x 2"	Box of 10 Pouches
AB-200300-003	2" x 3"	Box of 10 Pouches
AB-300300-004	3" x 3"	Box of 10 Pouches
AB-400400-001	4" x 4"	Box of 10 Pouches
AB-400500-005	4" x 5"	Box of 10 Pouches